

Assembly/Disassembly

Instruction for Instruments

Description: Rod to Rod Coupling

REF: 4922-1-010

Also applicable to the following Catalogue Numbers:


4922-1-020 / 4922-1-030

Manufacturer:

Stryker GmbH


Bohnackerweg 1, 2545 Selzach, Switzerland

Literature No: ADI0024/B

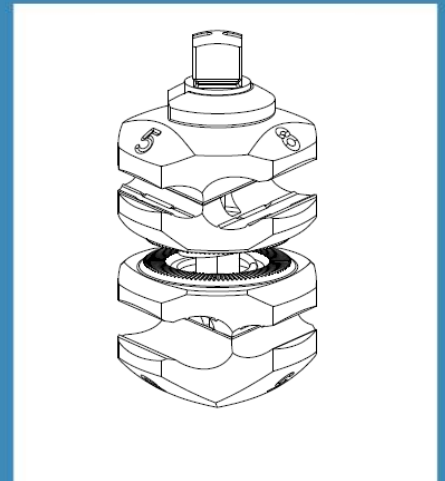


Disassembly Instruction


1. Open the coupling by turning the screw.

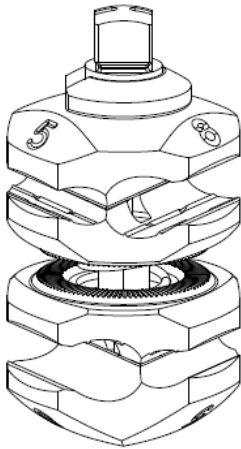

2. Align the slot in the washer with markings at screw head. These markings are an indicator for the position of two notches on the screw shaft which allow removal of washer only at this specific alignment.

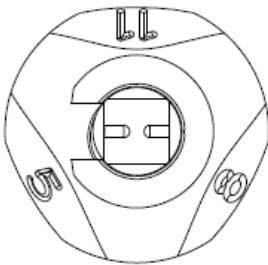


3. Compress the coupling.

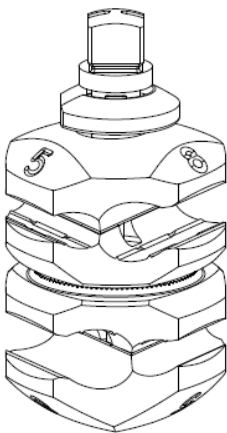


Disassembly Instruction

4. Remove the washer.

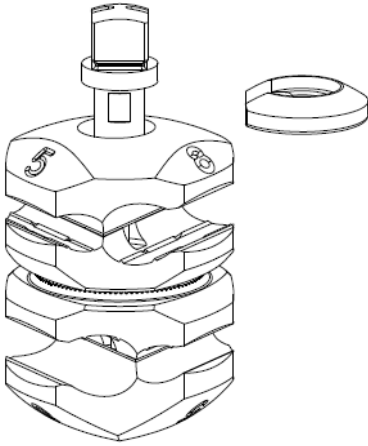


5. Release the coupling. The coupling is now ready to be cleaned.

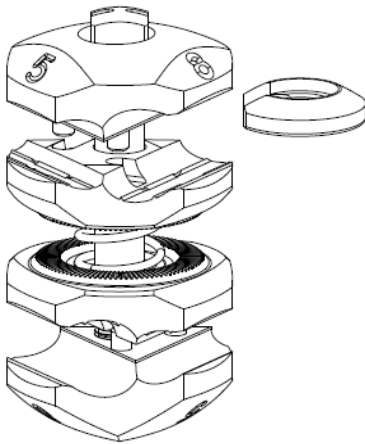


Assembly Instruction

1. Compress the coupling. Ensure that the coupling is fully unscrewed.

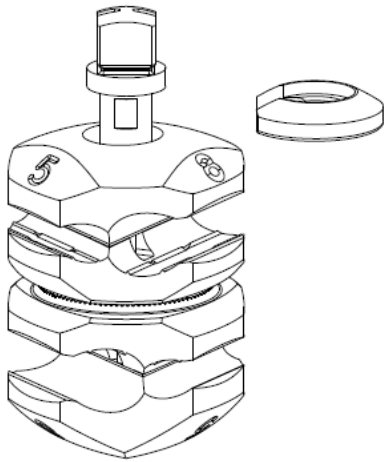

2. Insert the washer. Insertion of washer is only possible at the level of notches on the screw shaft.

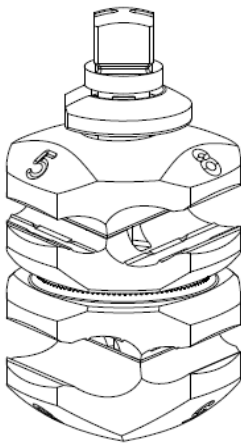

3. Release the coupling. The screw head must fit within the washer. Turn the screw 90° clockwise to prevent screw jamming during storage. The coupling is now assembled.

