

Assembly/Disassembly

Instruction for Instruments

Description: 5 Hole Pin Clamp

REF: 4922-2-020

Also applicable to the following Catalogue Numbers:

4922-2-220 / 4922-2-240 / 4922-2-320



4922-2-340

Manufacturer:

Stryker GmbH


Bohnackerweg 1, 2545 Selzach, Switzerland

Literature No: ADI0023/B

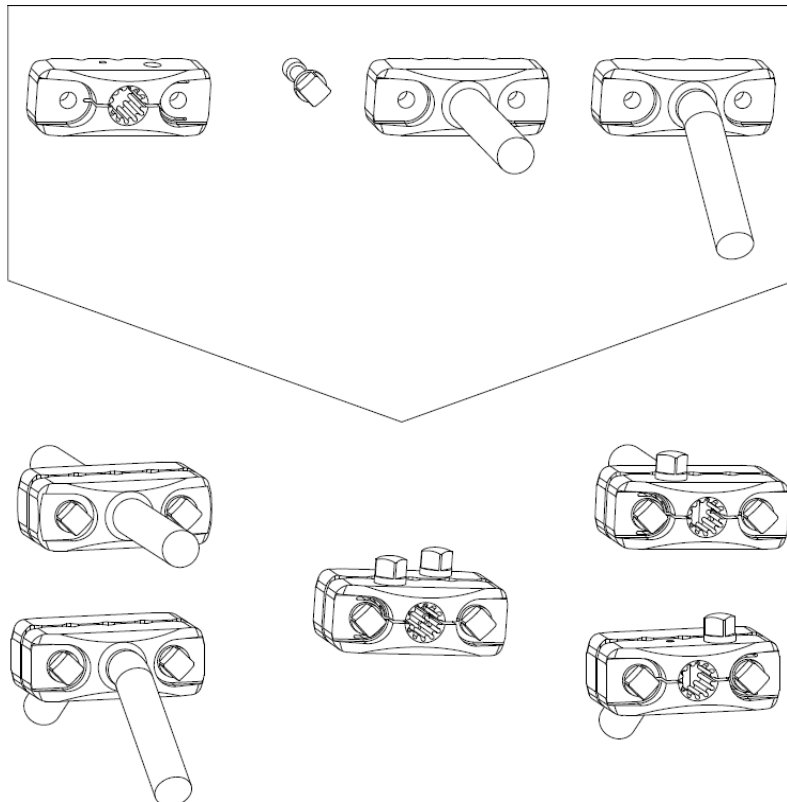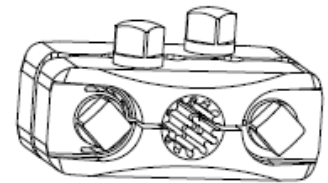


General Information

All configurations of the 5 Hole Pin Clamp consist of four interchangeable components.

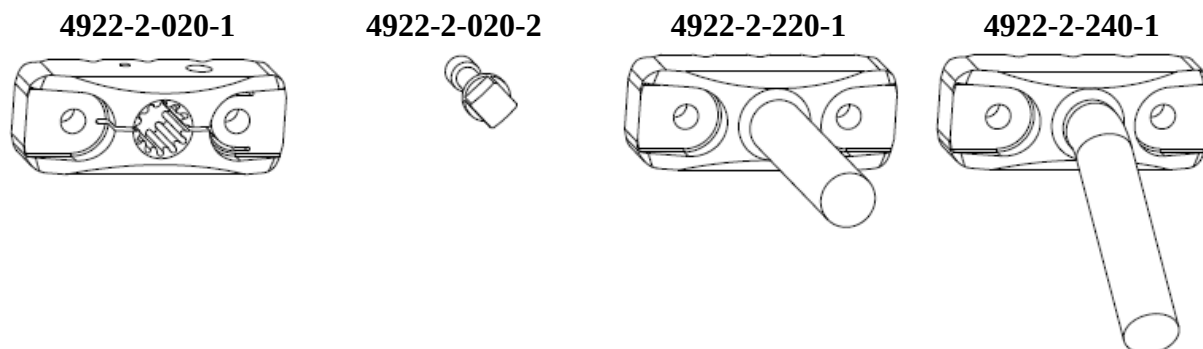


The following configurations of the 5 Hole Pin Clamp are available.

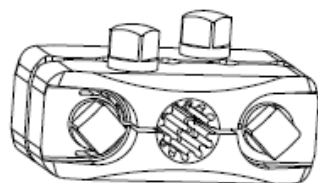


4922-2-020

consists of:
2 x 4922-2-020-1
4 x 4922-2-020-2

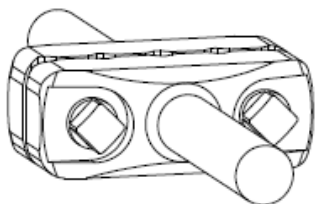


4922-2-220

consists of:
2 x 4922-2-220-1
2 x 4922-2-020-2

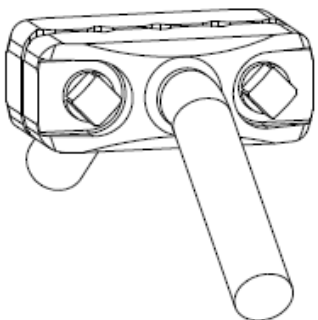


4922-2-240

consists of:
2 x 4922-2-240-1
2 x 4922-2-020-2

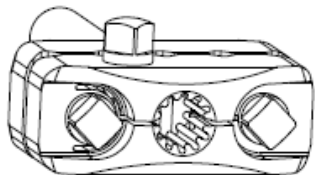

4922-2-320

consists of:
1 x 4922-2-220-1
1 x 4922-2-020-1
3 x 4922-2-020-2

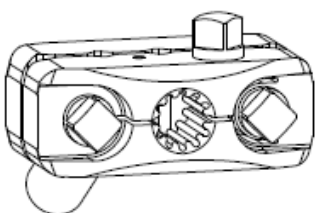

4922-2-340

consists of:
1 x 4922-2-240-1
1 x 4922-2-020-1
3 x 4922-2-020-2

Assembly / Disassembly Instruction

All 5 Hole Pin Clamp assemblies can be disassembled by removing the screw. As an example the 5 Hole Pin Clamp, REF 4922-2-020, is shown below.

