

Assembly/Disassembly

Instruction for Instruments

Description: Rod Coupler, 30°


REF: 4922-1-220

Also applicable to the following Catalogue Numbers:

Manufacturer:


Stryker GmbH
Bohnackerweg 1, 2545 Selzach, Switzerland

Literature No: 2012634 Rev. 1

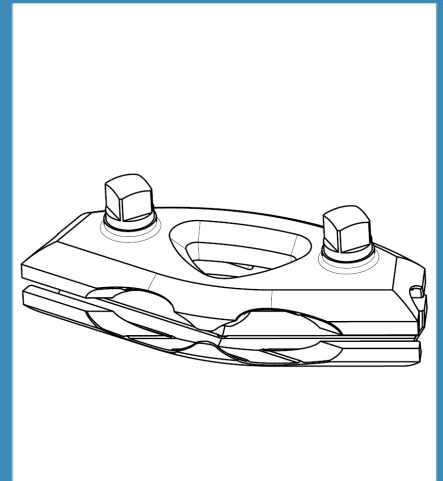


Disassembly Instruction

1. Fully unscrew and remove the two tightening screws.

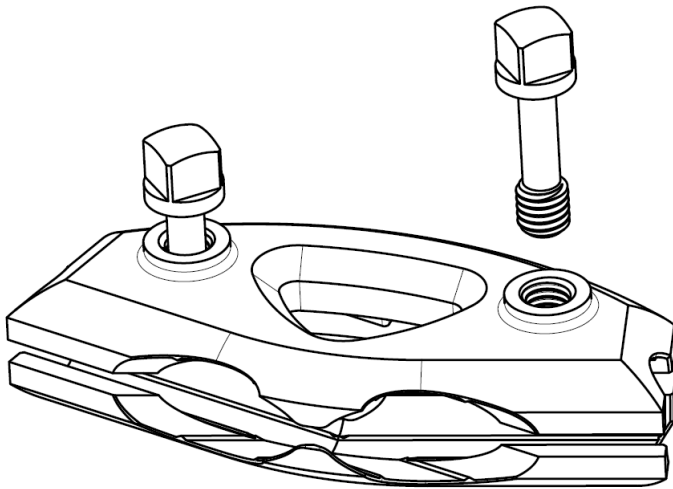


2. Separate the two coupling halves and the coupling is now ready to be cleaned.

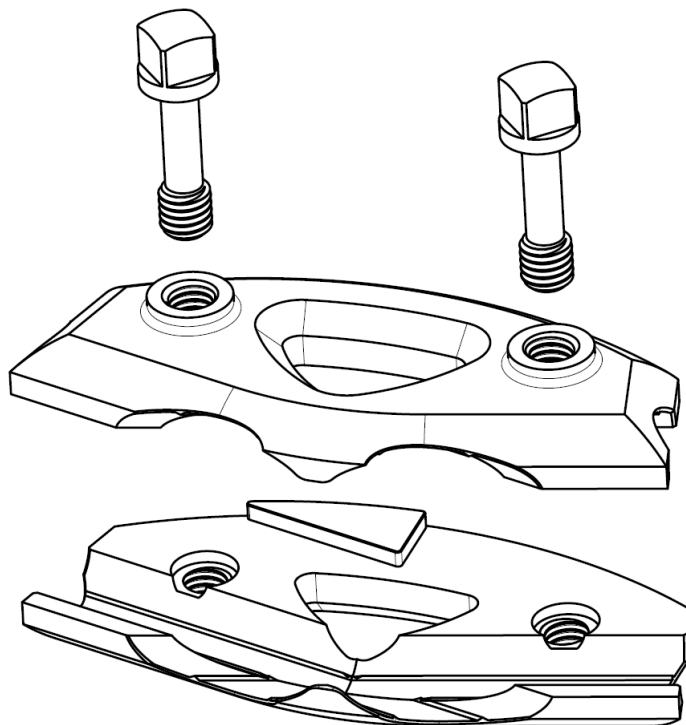


Assembly Instruction

1. Fully Insert the two tightening screw in the threaded holes of the first coupler flange.

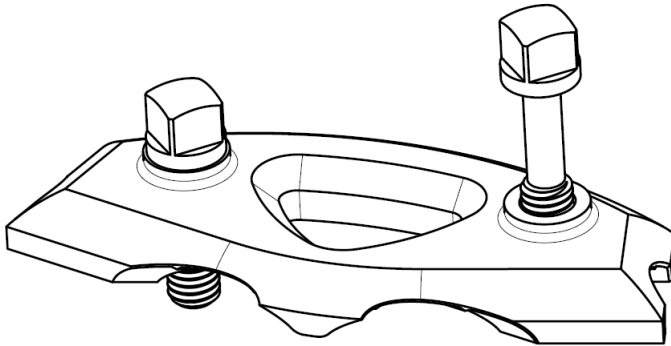

2. Add the second coupler flange and tighten it with the two tightening screws.

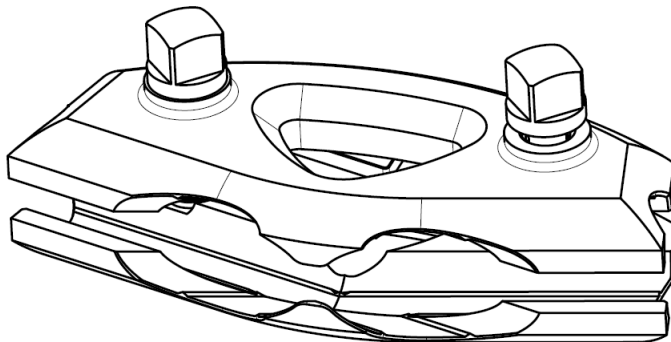

3. Untighten the screws 1-2 turns to ensure, that Ø11mm connecting rods fit into the rod coupler.

