

Trauma & Extremities

Case study: Use of Vitoss BA and VariAx Clavicle Locking Plate System for a Clavicle Nonunion

A review by Kenneth Koval, MD

Patient history:

Patient is a 65 year old female who sustained multiple injuries in a motor vehicle accident, including a tibial plateau fracture, cervical spine injury and a displaced distracted right clavicle fracture (Figure 1). Treatment of the cervical spine injury was prioritized and required a cervical orthosis, which could not be removed. Due to the cervical orthosis her clavicle fracture was treated non-operatively with a sling.

Assessment:

Patient returned at 5 months post injury with continued pain, tenderness, and deformity around her right clavicle. Radiographs demonstrated a clavicle nonunion (Figure 2). It was determined that she was a candidate for an open reduction and internal fixation.

Procedure/treatment:

Surgery was performed with an incision directly over the clavicle with a takedown of the nonunion, mobilization of the fragments, and stabilization with the VariAx Clavicle Locking Plate System Superior Lateral Plate. Vitoss BA, hydrated with blood from the surgical site, was used to graft the nonunion. (Figure 3A and 3B)



Figure 1: 65 year old woma s/p MVA. initial radiographs

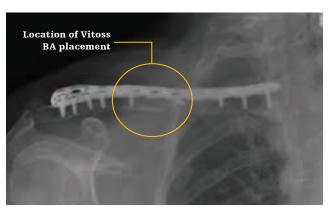


Figure 3A: Surgical treatment with Stryker plate and Vitoss BA



Figure 2: 5 months after injury

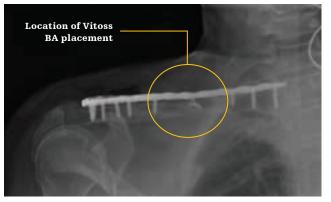


Figure 3B:



Post-operative clinical outcome:

Post-operatively the arm was placed in a sling and the patient was allowed to do pendulum exercises and passive assisted range of shoulder motion. At six weeks, the patient was allowed active shoulder motion and at 12 weeks resistive exercises.

Follow-up:

Patient was seen for 3 month (Figure 4A and 4B) and 6 month follow-up (Figure 5A and 5B) and did well clinically. The clavicle united uneventfully with the patient regaining full painless shoulder motion.

Conclusion:

This case illustrates a safe and effective use of the VariAx Clavicle Locking Plate System in combination with Vitoss BA, as a substitute for autologus bone graft. The use of Vitoss BA has shown to be a suitable 3-dimensional matrix to allow bone growth through the graft.

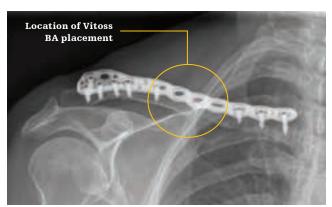


Figure 4A: 3 months after surgery

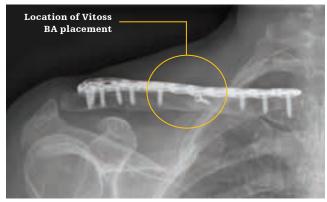


Figure 5A: 6 months post-op

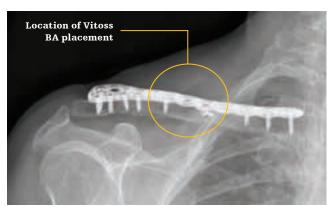


Figure 4B:

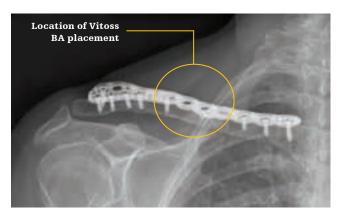


Figure 5B:

A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery. The information presented is intended to demonstrate the breadth of Stryker product offerings. A surgeon must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area. Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker, VariAx, Vitoss. All other trademarks are trademarks of their respective owners or holders.