

# LITE Midline Retractor

**EXPERIENCE** 

THE DIFFERENCE.

**EXPERIENCE** 

INNOVATION.

The Stryker LITe Midline
Retractor System employs a
uniquely shaped retractor blade
to sit directly atop the facet
capsules to facilitate a less
invasive midline incision.



 Retractor can be oriented cephalad/caudal.  System compatible only with Phantom Light Source (48252106) and Lateral Phantom Light Cable with Tedan Connector (48758451).







Left and right retractor blades sit atop the facet joint. Blade teeth are designed to enhance retraction during less invasive midline procedures.

### Retractor Blades

All LITe Midline Retractor Blades are radiolucent and excellent for procedures using fluoroscopy.



Cross cut connector is designed to prevent rotation of blade while attached to the blade handle.





Two Blade Configurations:
 Wide and Narrow
 Lengths: 40-120mm

### Blade Handle

The Side-Loading Retractor Blade Handle attaches to any retractor blade, creating a hand-held retractor that is used to place the blade into the operative site. A grooved handle is designed to enhance grip.





 Dual Blade Holder - Holds two blades per side to accommodate multilevel procedures.

## Lighting

 Phantom Light Source, Bright 52-watt LED light source, cool heat (48252106)



 Lateral Phantom Light Cable, 10 Ft, Strong Curve, with Tedan Connector (48758451)

### Standard Set

Reference	
Number	Description
48850005	Side-Loading Retractor Blade Handle
48850010	Midline Retractor
48850025	Depth Gauge
48850015	Dual Blade Holder
48850020	T-Handle
48851040	Retractor Blade, Wide, Left, 40mm, Red
48852040	Retractor Blade, Wide, Right, 40mm, Red
48851050	Retractor Blade, Wide, Left, 50mm, Violet
48852050	Retractor Blade, Wide, Right, 50mm, Violet
48851060	Retractor Blade, Wide, Left, 60mm, Gold
48852060	Retractor Blade, Wide, Right, 60mm, Gold
48851070	Retractor Blade, Wide, Left, 70mm, Light Green
48852070	Retractor Blade, Wide, Right, 70mm, Light Green
48851080	Retractor Blade, Wide, Left, 80mm, Turquoise
48852080	Retractor Blade, Wide, Right, 80mm, Turquoise
48851090	Retractor Blade, Wide, Left, 90mm, Fuchsia
48852090	Retractor Blade, Wide, Right, 90mm, Fuchsia
48851100	Retractor Blade, Wide, Left, 100mm, Chocolate
48852100	Retractor Blade, Wide, Right, 100mm, Chocolate
48853040	Retractor Blade, Narrow, Left, 40mm, Red
48854040	Retractor Blade, Narrow, Right, 40mm, Red
48853050	Retractor Blade, Narrow, Left, 50mm, Violet
48854050	Retractor Blade, Narrow, Right, 50mm, Violet
48853060	Retractor Blade, Narrow, Left, 60mm, Gold
48854060	Retractor Blade, Narrow, Right, 60mm, Gold
48853070	Retractor Blade, Narrow, Left, 70mm, Light Green
48854070	Retractor Blade, Narrow, Right, 70mm, Light Green
48853080	Retractor Blade, Narrow, Left, 80mm, Turquoise
48854080	Retractor Blade, Narrow, Right, 80mm, Turquoise
48853090	Retractor Blade, Narrow, Left, 90mm, Fuchsia
48854090	Retractor Blade, Narrow, Right, 90mm, Fuchsia
48853100	Retractor Blade, Narrow, Left, 100mm, Chocolate
48854100	Retractor Blade, Narrow, Right, 100mm, Chocolate
48850040	Sterilization Tray
48758451	Lateral Phantom Light Cable with Tedan Connector
48252106	Phantom Light Source
48851313A	Disposable LITe Cable
48851313B	Universal LITe Cable Adaptor

#### **Outlier Blades**

Reference Number	Description
48851110	Retractor Blade, Left, 110mm, with Light Guide, Turquoise Splash
48852110	Retractor Blade, Right, 110mm, with Light Guide, Turquoise Splash
48851120	Retractor Blade, Left, 120mm, with Light Guide, Purple Splash
48852120	Retractor Blade, Right, 120mm, with Light Guide, Purple Splash
48853110	Retractor Blade, Narrow, Left, 110mm, with Light Guide, Turquoise Splash
48854110	Retractor Blade, Narrow, Right, 110mm, with Light Guide, Turquoise Splash
48853120	Retractor Blade, Narrow, Left, 120mm, with Light Guide, Purple Splash
48854120	Retractor Blade, Narrow, Right, 120mm, with Light Guide, Purple Splash



Distributed By: Stryker Spine 2 Pearl CT. Allendale, NJ 07401 USA

Stryker Spine SA LE CRET DU LOCLE 10A 2300 LA Chaux De Fonds Switzerland

P: +1-201-760-8000

ONET, BV Hommerterweg 286

P: +1-713-726-0886

Manufactured By

**TEDAN Surgical** 

Innovations, LLC 12815 W. Airport BLVD

EC REP

Suite #200 Sugarland, TX 77478

6436 AM Amstenrade The Netherlands



Manufactured By Sunoptic Technologies(R) 6018 Bowdendale Avenue Jacksonville, FL 32216 USA

EC REP

RMS UK, Ltd. 28 Trinity Road Nailsea, Somerset BS48 4NU **United Kingdom** 

A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A surgeon must always refer to the package insert, product label and/or instructions for use before using any Stryker product, or OEM product distributed by Stryker. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: LITe, Phantom, Stryker. All other trademarks are trademarks of their respective owners or

PRTTL-SS-1\_Rev-1 SC/GS 01/16